



Royal Dental
Manufacturing, Inc.
12414 Highway 99
Everett, WA 98204-5548
Tel. (425) 743-0988
Fax (425) 743-3588

Dear Doctor,

Royal Dental chair upholstery selected from the Royal Standard Color Guide is protected with a special surface coating that resists stains and discolorations.

CARE AND CLEANING INSTRUCTIONS FOR ROYAL DENTAL CHAIRS

Buildups of disinfectant are known to harm the upholstery's protective coating as well as the chair's other surfaces.

Using barriers to control infection in lieu of dental surface disinfectants will most effectively increase the longevity of all of the surfaces on the chair.

If you must use disinfectants then clean the chair afterwards as described below.

NOTE: The longer a stain is allowed to set, the more difficult it will be to remove.

NOTE: Use damp, not wet, cleaning cloths around control switches and other openings. Do not allow liquid to run into the interior of the chair.

For Daily Cleaning:

- Use a solution of 10% household liquid dish soap mixed with warm water and apply with a soft damp cloth to remove most soiling and buildups. If necessary, rinse, re-dampen the cloth and continue working out remaining residue. Then wipe the surface clean with a soft clean water-dampened cloth.
- Formula 409 or Fantastik spray cleaners may also be used. Be sure to rinse with a soft clean water-dampened cloth afterwards. **Do not use Citrus, Bleach or other variations of these cleaners, only the original formulas.**

For more difficult stains not solved by above method:

- Dampen a soft white cloth with a standard 70% formulation of Isopropyl (rubbing) Alcohol and rub an inconspicuous area of the material first, insure that no discoloration has occurred, and then rub the stained area gently 6 times. **Note: Use extreme caution with this method. Complete only in a well-ventilated area and away from any open flame.**
- Rinse with a water-dampened cloth.

Do not use Acetone (finger nail polish remover), Sodium Hypochlorite (bleach), Naphtha (lighter fluid) or D-limonene (citrus cleaners). These are known to damage the chair's finish.