



MODIFIED ARMREST FRAME INSTALLATION

1. Remove the chair seat cushion and upholstered skirt to expose the armrest frame socket on the main chair frame.
2. Remove the steel pin from the armrest frame.
3. Insert the armrest frame into the armrest frame socket. Line up the pinhole with the slot in the socket and insert the pin.
4. Rotate the armrest frame to expose the small setscrew and tighten the setscrew to secure the pin in place.
5. Re-install the skirt and seat cushion.

Please contact Royal Technical Service if you have any questions or experience any difficulties with this procedure.