
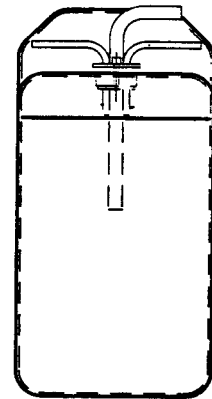


Checking the Hydraulic Level:

- 1) Lower Base to full down position. Lower Backrest to full down position (see illustration below).
- 2) Remove 3 allen screws in lower front cover (using 3/32" allen wrench) and put front cover aside.
- 3) Visually check reservoir fluid height (see illustration below). If fluid level is low see filling procedure below. If fluid level is acceptable, reinstall front cover.

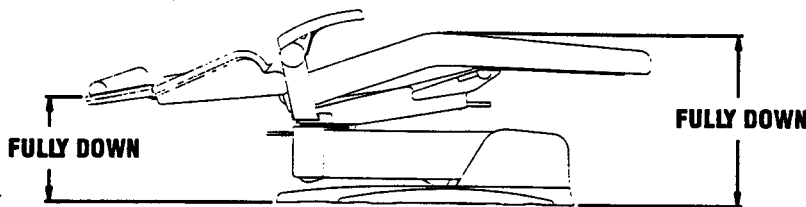

Filling Procedure:

- 1) Working at the front of chair loosen reservoir plug allen screw by 6 turns (max.) (see illustrations below).
- 2) Pull suction tube assembly up for access to reservoir opening.
- 3) Fill reservoir to level indicated in sketch (see above) with ISO46 grade Hydraulic Oil.
- 4) Reinstall suction tube assembly, retighten allen screw on plug, and reinstall front cover.

